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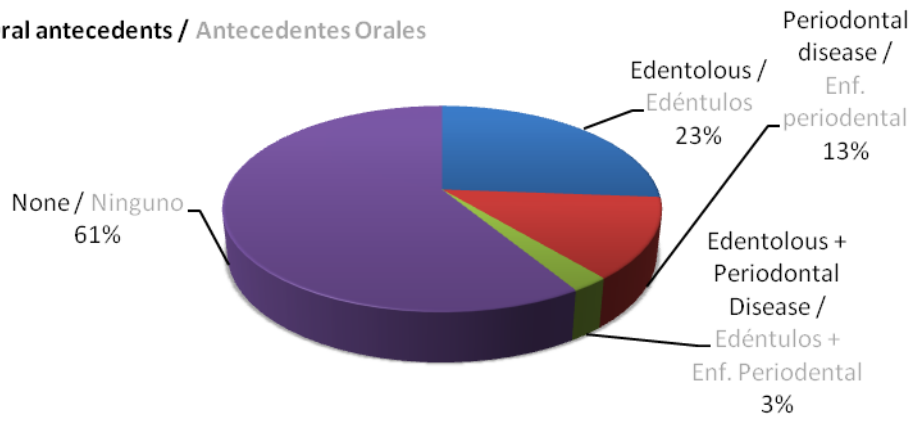
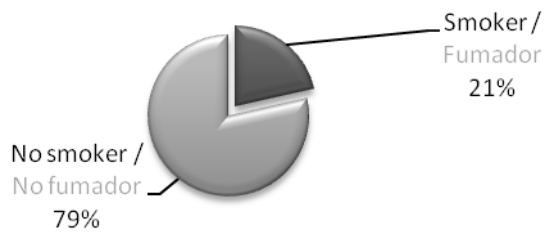
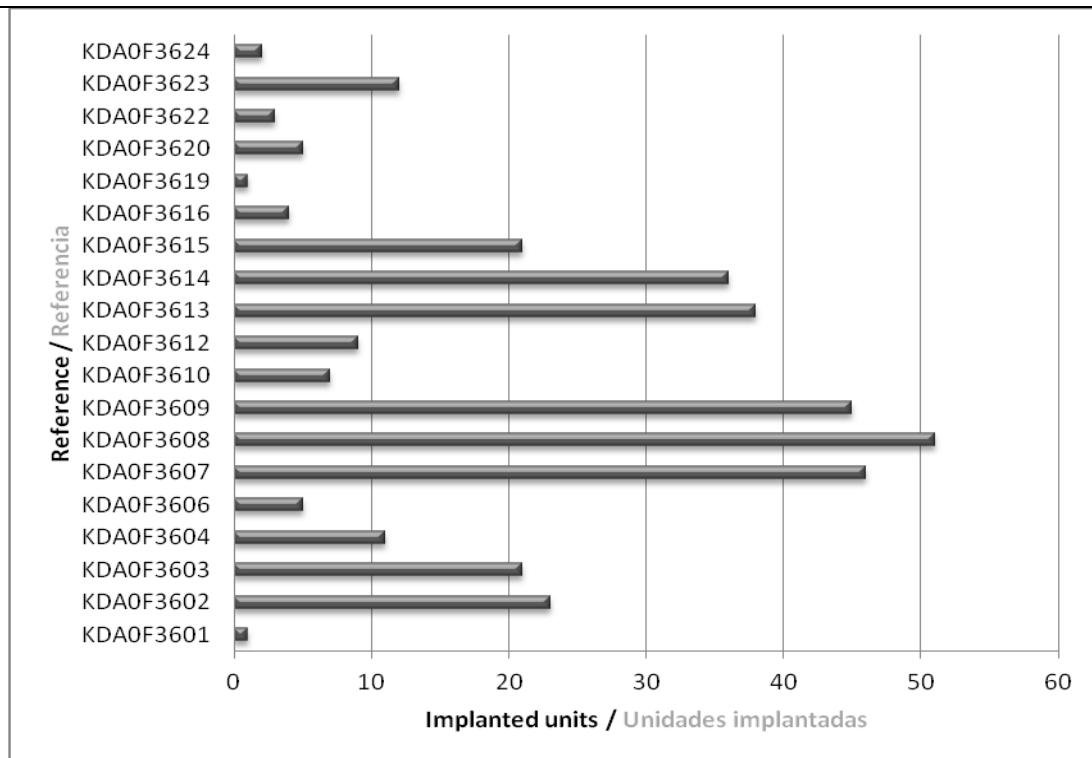
Medical antecedents: Others / Antecedentes Médicos: Otros

Range of implants / Gama de implantes

The study was performed about the internal hexagon dental implants manufactured by the company Ilerimplant s.l., the following references are included in the study:

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
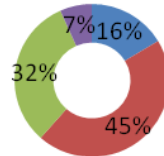
Lenght

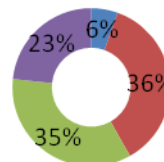■ L8 ■ L10 ■ L11,5 ■ L13

References shaded in light blue are the measures most frequently used. Practically, the entire range of implants of the company are covered with data reported as you can see, except the Ø5.75 diameter implants due any follow-up have not received since its use is relegated to very specific cases. Even so, their inclusion for the correct evaluation of the implants response are not considered essential, since its characteristics are equivalent to the implants of different diameters, and therefore it can be expected a similar response.

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The type of surgery used for the 351 implant placements was:

130 implants were placed through a single surgery and 156 were placed with 2 surgeries as per our recommended surgical technique. We have no information on the remaining 20% (65 implants), but 21 implants were placed after extraction and none of them were charged immediately.

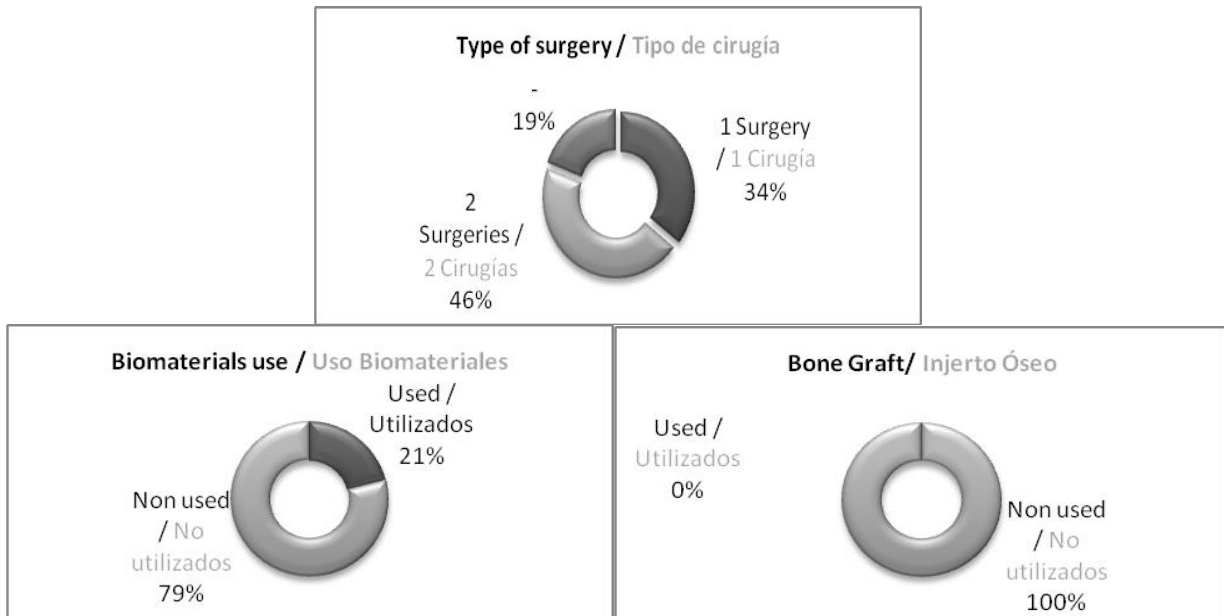
Biomaterials and bone graft were used to help the implant integration in 21% and 0% of the interventions respectively.

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
Regarding loaded implants, the rehabilitation of 130 implants was carried out through a single crown, 175 were rehabilitated through a fixed bridge, 21 with ball pillars overdenture and 21 with bar overdenture, we have no information about 4 implants.

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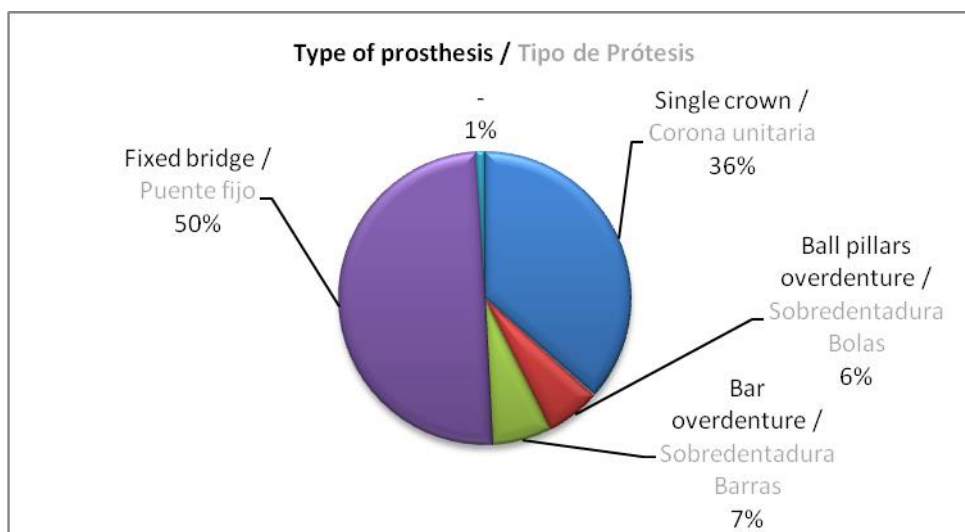
Functional loading for 321 and 20 implants was classical and immediate respectively, except 2 early loadings. While for the remaining 12, there are 6 implants which were not indicated the type of functional

loading received (1 of them was explanted by no-osseointegration). We have lost following of remaining 6.

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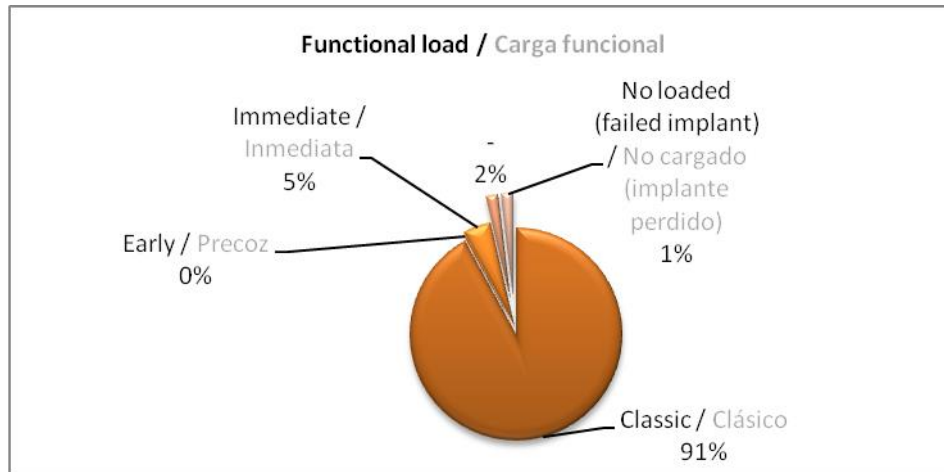
Regarding implants without information we do not know the exact details of placement about 11 of them

which their information is not disposal. About remaining 18, we know that 8 were placed into the maxilla and the remaining 10 in the jaw.

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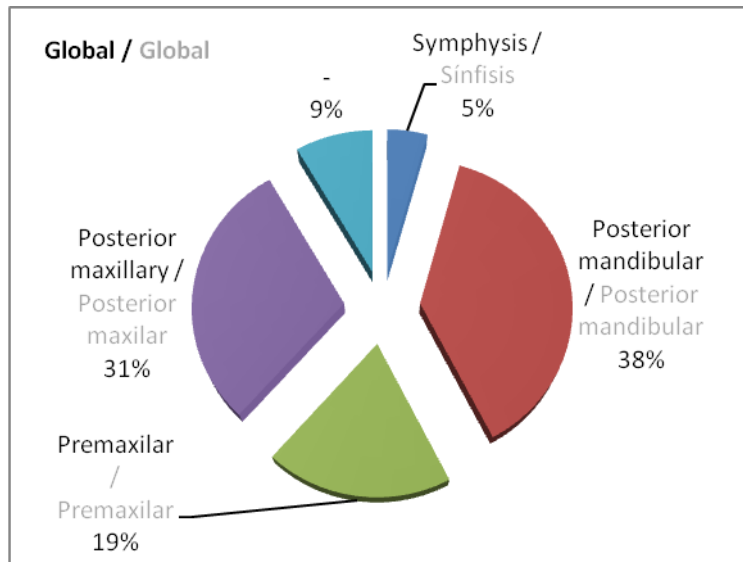
4 failed implant (2%)

Premaxilla: 3 failed implant.

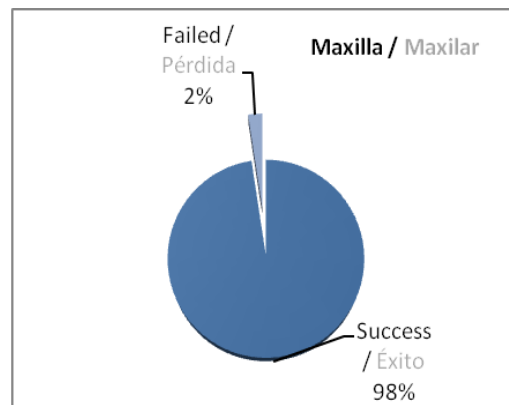1. Patient description: 67 years old and edentulous, 4 implants were placed, three of them were rehabilitated through a fixed bridge, one of them was failed (Implant placed at position 23 (Canine tooth) (KDAF3608 – Ø3.75 / L11.5) (2 surgeries).
2. Patient description: 57 years old with periodontal disease, 2 implants were placed, one of them was failed on second surgery (Implant placed at position 23 (Canine tooth) (KDAF3604 – Ø3.30 / L13) (2 surgeries).
3. Patient description: 43 years old, smoker and edentulous, 2 implants were placed, one of them was failed (1 surgery). Remark: The patient suffered for unknowable diabetes after the placement of the first implant.

Posterior maxilla: 1 failed implant.

4. Patient description: 39 years old, smoker, 8 implants were placed, one of them was failed for lack of osseointegration after 4 months (Implant placed at position 17 (2nd molar) (KDAF3608 – Ø3.75 / L11.5) (1 surgery). We must note that this implant was not placed properly based on occlusal diagram of the company and it would suffer a fatigue failure surely. We can not

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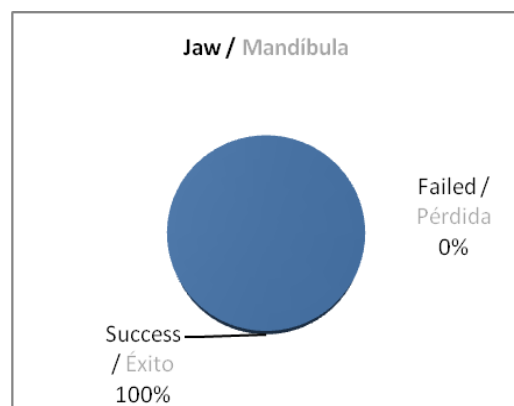
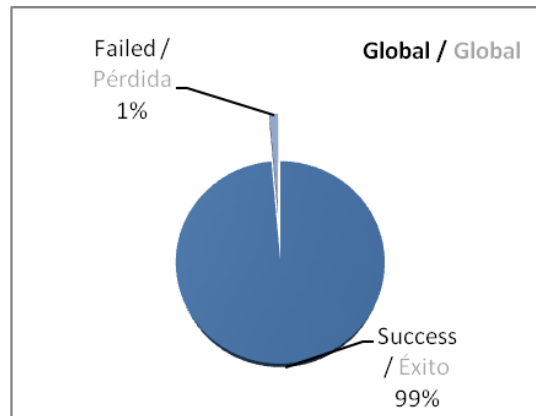
GLOBAL SUCCESS RATE / TASA DE ÉXITO GLOBAL: 99%

CONCLUSIONS / CONCLUSIONES

Currently, based on the data analyzed, Frontier model implants of Ilerimplant S.L. company (GMI trademark) have a surgical rate similar to the rest of the successful implants available with similar features in the market. Remember that these results are only short-term success rates (placement-loading).

Referring to the four implants lost we must indicate that were not implanted as per recommended occlusal diagram and validated by the company, which indicates that the diameter used for a canine tooth should be at least 4.25 mm, so the implants 1 and 3, it could consider as a possible cause of implant failure, an implant overload which prevent a proper osseointegration.

Frequency of use of different measures of implants according to this study:

- The diameter most widely used: 3.75 mm (45%)
- The length most widely used: 10 and 11.5 mm (both with 36 and 35%, respectively)

Based on the results obtained it does not considered to open any corrective and/or preventive action. We hope in the following reports have more cases to study, but by the time, according to 351 cases analyzed it is showed that our implant works properly and that its success rate is excellent and considering that in failed implants, patients with unfavourable antecedents for an implant placement were encountered in 39% cases.

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❖ **REPORT 3: LONG-TERM FOLLOW-UP / INFORME 3: SEGUIMIENTO A LARGO PLAZO**

After evaluation of the proper osseointegration and implant charging in the short-term follow-up, where 4 of 351 implants failed during the osseointegration stage, inducing 99% of success rate, a long-term product follow-up has been developed, and the outcomes obtained by the time are attached below there are:

Regarding the 341 implants charged properly, it has lost the follow-up of 66 (25 of 135 patients), due to patients have not attended established revisions by dentist, this involves 20.66% loss of sample follow-up.

The rest of 275 implants work properly as per information provided by doctors. Obtaining a 100% of success rate in the second stage of the study for those implants has performed the follow-up.

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*Patient with both succeeded and failed implants. / Pacientes con implantes exitosos y fracasados.

Table 1 – Correlation between follow-up time and number of implants.

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Must be also mentioned that the results obtained in the study performed by University of Sevilla were attached in the long-term follow-up report n° 1 issued on 01 / 09 / 2014.

- **Early loading of GMI implants with internal connection and sandblasted-acid-etched surface.**
Velasco Ortega E., Monsalve Guill I., Matos Garrido N., Jiménez Gerra A., García Méndez A., Medel Soteras A., Ortiz García I., España López A.

Published in the *AVANCES en Periodoncia e Implantología* (Volume 26 – Number 2 – August 2014).

Study results: After 1 clinical follow-up year and implant post-loading, the success rate of the GMI Frontier implants was 99.1%.

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