

dental wings



Instructions to import
gmi libraries

into



dental wings

software

Download software

From your computer connect to **www.gmidental.com**

Go to **PRODUCTS** menu, open the **CAD/CAM Library > INTERFASE** or **DIRECT TO IMPLANT** section and select **Dental wings**.


Click on  dental wings icon to start the download.

A file (*with the updated revision number*) will be downloaded into *downloads folder* of your computer.

Import library


Import the library into DWOS.

Select in the menu **Application -> Implant Kits**.

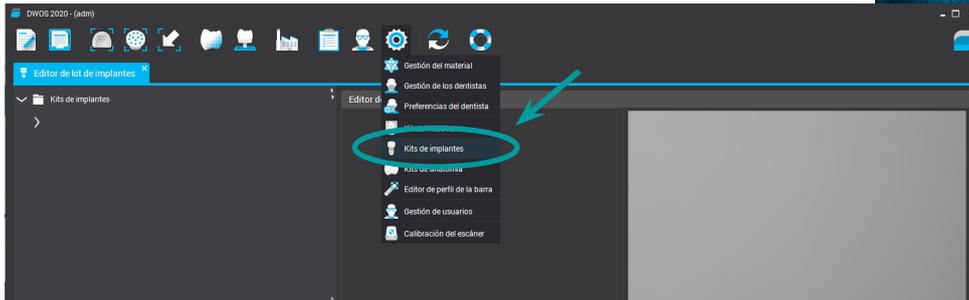

Import library


Place the cursor on "*Implant Kit*" and press the right button.
In the drop-down menu select "*Import kits from implant manufacturer*".

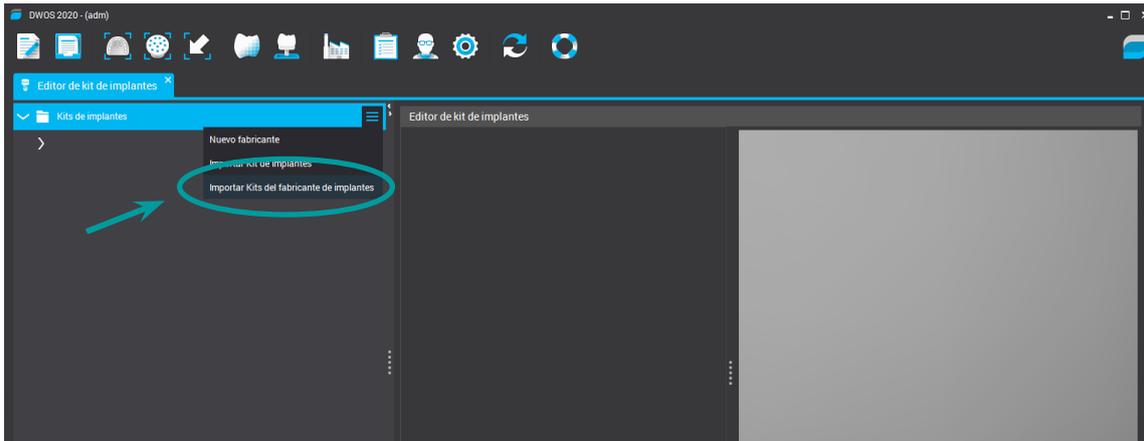

Import library


A window will appear to choose the file. Choose the library that you have downloaded from the web and that will be found in the *Downloads* folder.

Select "Open".

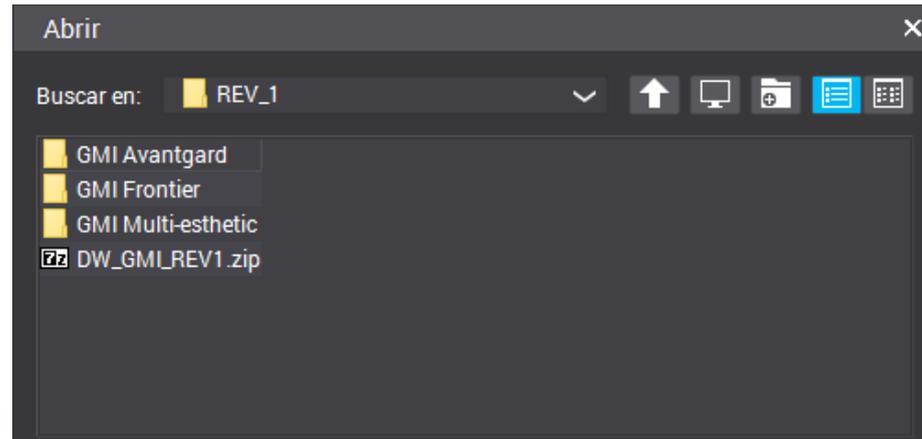



Import library

When the library is exported successfully you will see that the **gmi** implant library is added.

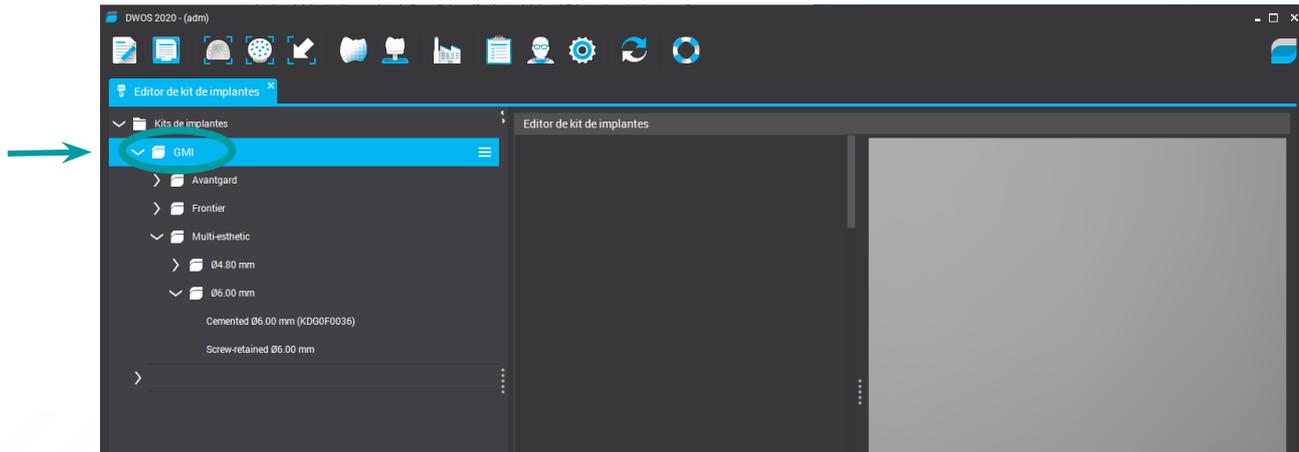

Import library

When you select a kit, the screen shows the list of all the components of that kit that can be activated for viewing.

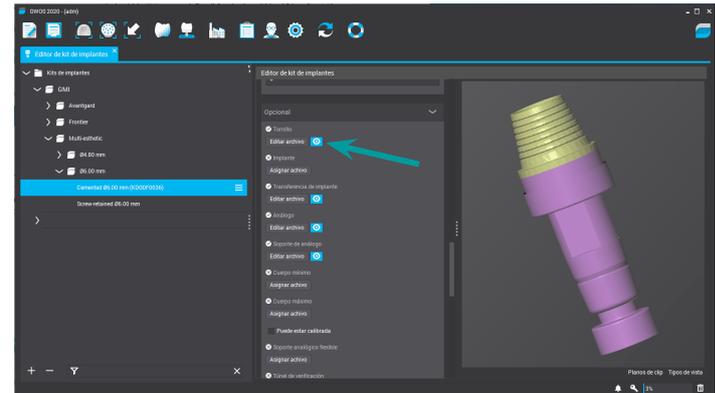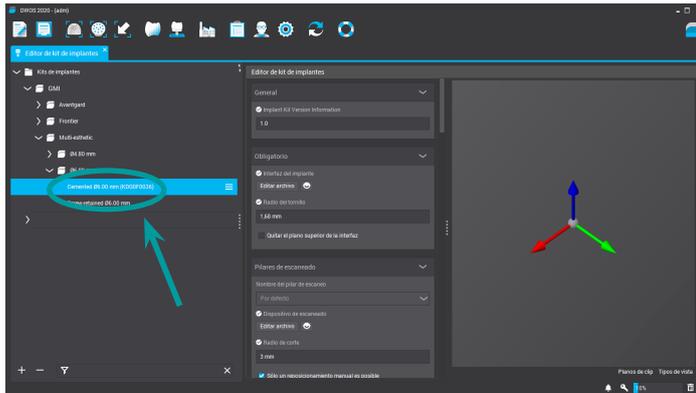

Import library

Now you can choose the connection you want to design.

Thank you for trusting us

