3shape

Instructions to import
gmi libraries
into 3shape  software

Download software

From the computer connect to **www.gmidental.com**

Go to **PRODUCTS** menu, open the **CAD/CAM Libraries > Interfase** or **Direct to implant** section and select **3Shape**.

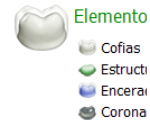
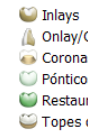
Click on **3shape**  icon to start the download.

A file (*with the updated revision number*) will be downloaded into *downloads* folder of your computer.

Import library


The library is imported using the *Control Panel* of 3shape.

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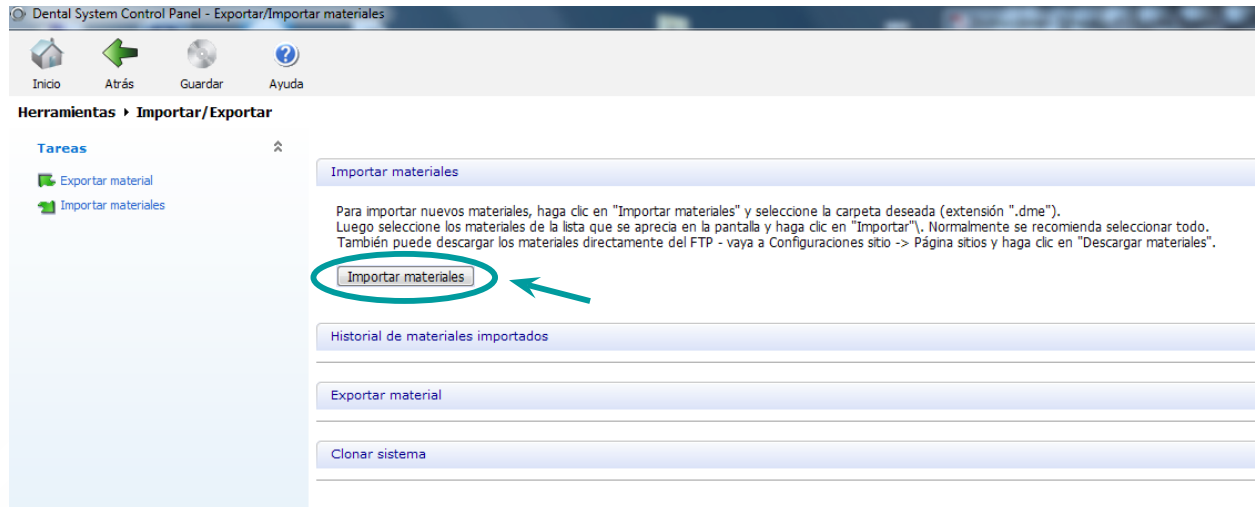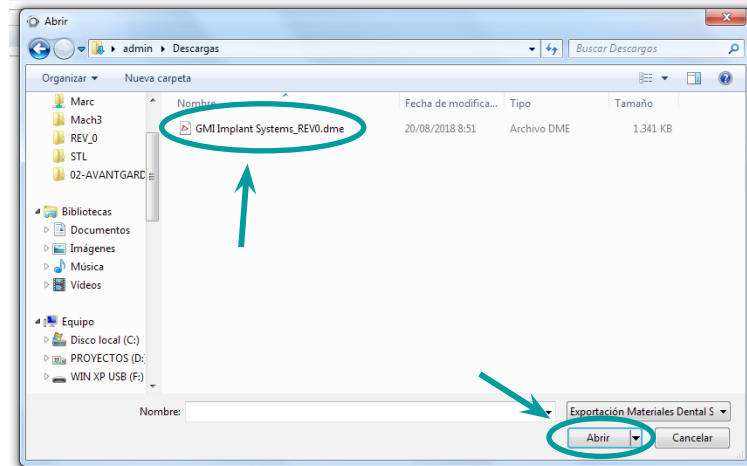
Clonar sistema

Import library


A window will appear to choose the file. Choose the library you have downloaded from the web in the *downloads* folder.

Choose "Open".

Import library

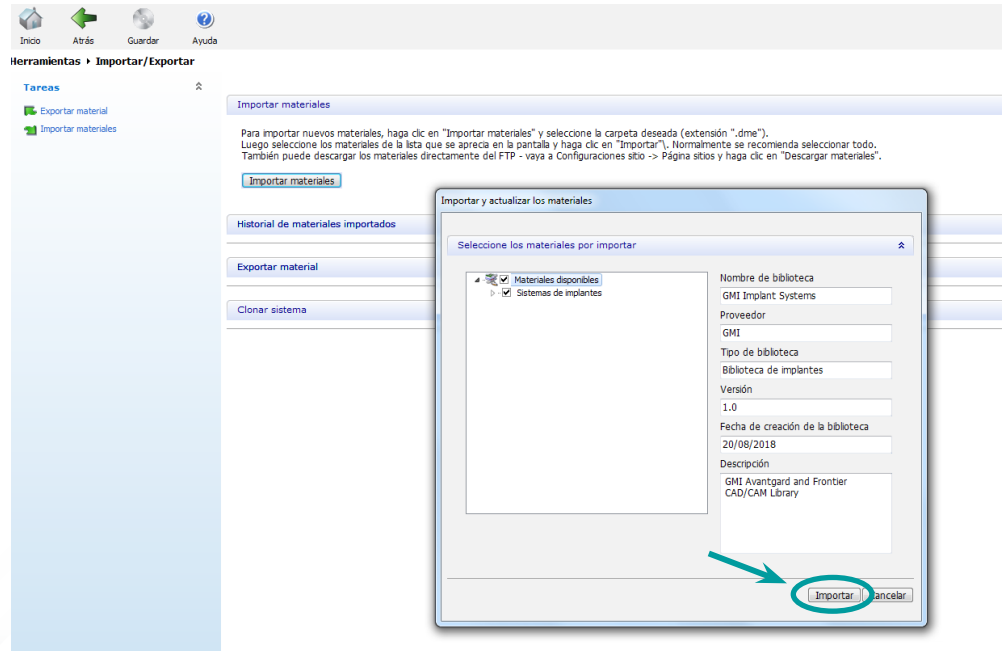
Select "Import".

The screenshot shows the GMI software interface. At the top, there are navigation icons: Inicio, Atrás, Guardar, and Ayuda. Below these, the menu 'Herramientas' is expanded to show 'Importar/Exportar'. Under this menu, there are options: 'Tareas', 'Exportar material', and 'Importar materiales'. The 'Importar materiales' option is highlighted. A dialog box titled 'Importar y actualizar los materiales' is open, showing a tree view of 'Materiales disponibles' with 'Sistemas de implantes' selected. To the right of the tree view, there are input fields for: 'Nombre de biblioteca' (GMI Implant Systems), 'Proveedor' (GMI), 'Tipo de biblioteca' (Biblioteca de implantes), 'Versión' (1.0), 'Fecha de creación de la biblioteca' (20/08/2018), and 'Descripción' (GMI Avantgard and Frontier CAD/CAM Library). At the bottom of the dialog box, there are two buttons: 'Importar' and 'Cancelar'. A red arrow points to the 'Importar' button, which is circled in red.


Import library

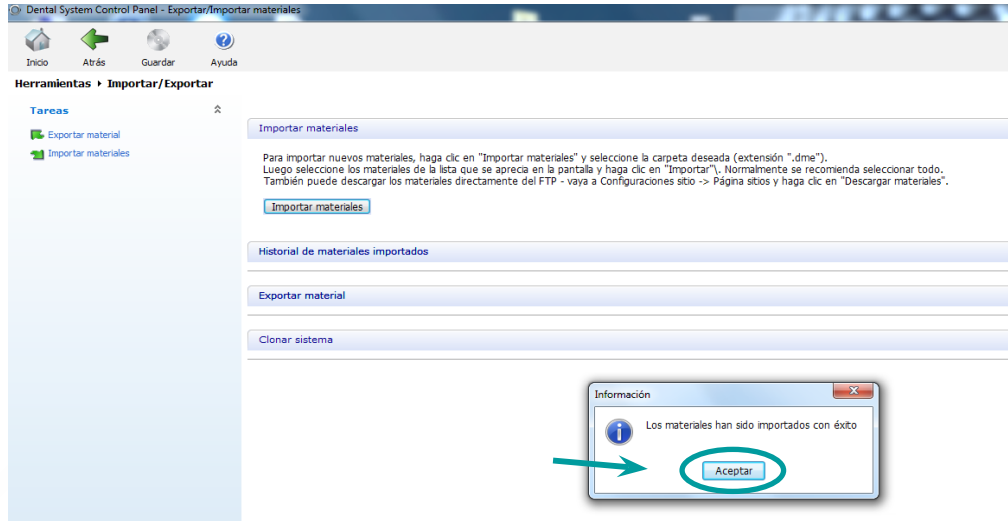Confirm it has been imported correctly.

Import library

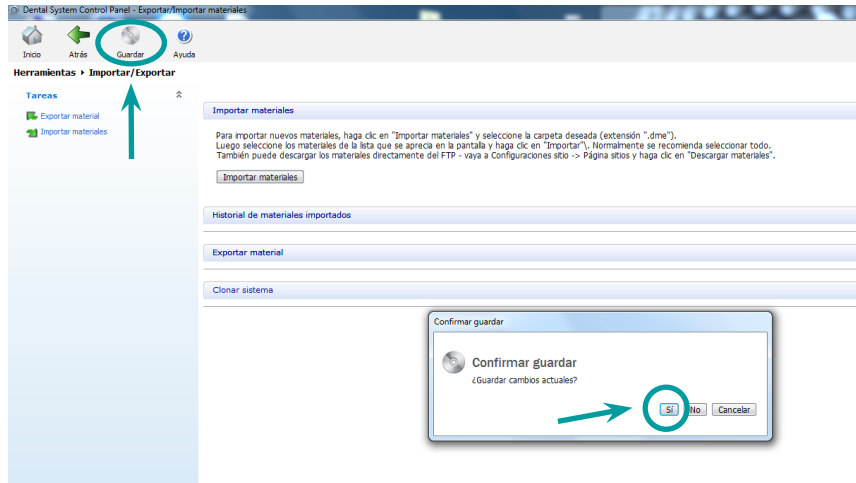Save, confirm and you can use the **gmi** libraries.

Thank you for trusting us